

OLYMPIC WARM-LAMPS™

*Two warming lamp options
provide safe, gentle
warmth for babies*

**Gentle Warmth for Changing,
Bathing, Feeding, Weighing,
Examinations, & Procedures**



Adapts to Many Uses

With its adjustable height & light hood, the Olympic Warm-Lamp is easy to set at the correct distance and angle for use during bathing, feeding, weighing & examinations. The control panel is set at just the right height for use with an Olympic Smart Scale or an examination table.



Radiant Heat & Light

Uses a special single bulb that furnishes both radiant heat & clear white light



Infrared Heat

Radiates invisible infrared heat, creating a wide zone of comfortable warmth – available with or without fully adjustable exam light

OLYMPIC WARM-LAMPS™



Adjustable Light Hoods:
Light hoods on all Warm-Lamps swivel and tilt and hold their position, providing complete control over direction of warmth.



Distance Measurer:
All Warm-Lamps have a retractable measurer on the side of the Light Hood to ensure warmth is the optimal distance from the baby.



Adjustable Height:
All models have adjustable height – from 58 to 74 in. (147.3 to 187.9 cm) – so the lamp can be placed at just the right distance from the baby.



Fully Maneuverable:
Warm-Lamps have a roll-around base with five swivelling casters for ease of movement, and are counter-balanced for stability & safety.



Easy-to-use Controls:
A 15 minute on/off safety timer and an adjustable intensity control are standard features on Olympic Warm-Lamp models.

Ordering Information

OLYMPIC WARM-LAMP™

Model 31 – Radiant Heat with Light

Roll-around floorstand model. Adjustable height. Warmth with incandescent light.

CAT. NO. 56831

Model 41 – Infrared Heat without Light

Roll-around floorstand model. Adjustable height. Infrared warmth without light.

CAT. NO. 56841

Model 45 – Infrared Heat with Exam Light

Roll-around floorstand model. Adjustable height. Infrared warmth with examination light.

CAT. NO. 56845

Electrical Safety: CSA C/US certified to CSA C22.2 601.1 M90 and UL 60601-1

Power Requirement: 120V~ 60Hz

natus®

Natus Medical Incorporated
1501 Industrial Road • San Carlos, CA 94070 USA
1-800-303-0306 • 1-650-802-0400
www.natus.com

For information or ordering
Call 1-800-303-0306
(toll-free in US/Canada)