

OLYMPIC STERILE-DRIERS™

*Bio-Clean Air for Safer
Drying & Packaging*

Model 44



Model 43



- Fast Drying
- HEPA Filtered Air
- Flexible & Convenient Loading
- Large Capacity



OLYMPIC STERILE-DRIERS™

- Speeds drying
- Protects the load while drying
- Provides short-term storage

Olympic Sterile-Driers offer faster drying in bio-clean air, as recommended by infection control authorities. For safety & efficiency, they are the preferred driers following pasteurization or other methods of high-level disinfection.

By the use of HEPA-filtered air, Olympic Sterile-Driers eliminate the threat of airborne pathogens recontaminating wet, warm parts while they are being dried. All air entering the drying chamber first passes through a pre-filter and then a HEPA filter which is 99.97% efficient in removing all particles larger than 0.3 micron.

And Sterile-Driers can dry more parts faster. They can handle large loads with almost any combination of tubes & parts. Faster drying is achieved by Olympic's exclusive recirculating air system that efficiently recaptures heat instead of dissipating it.

These & many other useful features have made Olympic Sterile-Driers the first choice for bio-clean air drying of respiratory and anesthesia breathing tubes and parts.

RECIRCULATING HEAT AND MA


- Pre-Filters**
Removes larger airborne particles. Front-mounted for easy replacement.
- HEPA Filters**
Separate HEPA filter for each drying chamber means greater air flow and longer filter life.



- Microprocessor-based temperature control.
- Digital Countdown Timer.
- Continuous or Timed Operation.



HEPA Filters

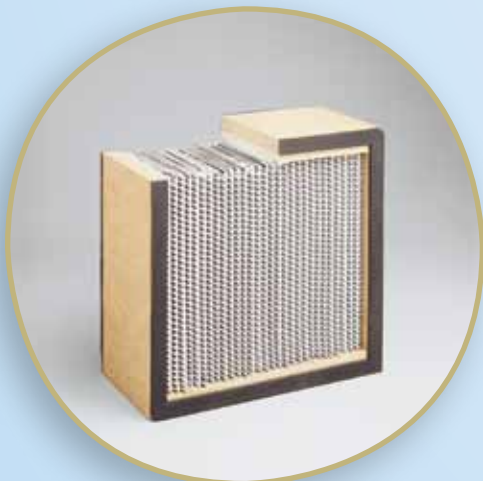

Certified class 100 High Efficiency. Particulate Air (HEPA) filters remove 99.97% of all particles larger than 0.3 microns that transport bacteria and viruses. Individually tested to assure entire HEPA filter system is leak-free.

Removeable Trays



Model 44 has nine large 18" x 18" (46 x 46 cm) parts trays standard. Model 43 has three 18" x 18" (46 x 46 cm) trays and six 18" x 9" (46 x 23 cm) half trays standard. Nylon coated to be cool to touch.

ANY OTHER EXCLUSIVE FEATURES



Separate Blowers
Each drying chamber has own blower for quieter operation & greater air volume.

Heaters
Thermostatically controlled.

**Recirculating Heat—
For Faster Drying**
More than 80% of heated air captured & recirculated through HEPA filters results in faster, more efficient drying. Typical time is 40 minutes.

Forced Vertical Airflow
Tubes dry faster because air is forced through interior lumen of each tube.

See-Thru Doors
On all models.

Rust-Proof
All aluminum construction.

Outside Vent Attachment
(Optional, not shown).

Model 44



Large drying capacity can handle full load from an Olympic Pasteurmatic. Model 44 shown with extra trays.

Model 43



Fast drying of tubes and parts.

Bag Drier



Optional Bag Drier solves the problem of drying both the inside and outside of breathing bags after pasteurization. Slides into any Sterile-Drier model manufactured since 1984 and dries up to 16 bags at once.

Slide-Out Tube Holder



Easy-loading tube-holders slide out at convenient height. Holds 35 tubes up to 5 ft. (1.5 m) long. Tube-holders can be replaced with more parts trays or Bag Drier.

More Good Reasons for Selecting Olympic Sterile-Driers

Safety Features

Over-temperature thermal cut-offs on heaters and blower motors. Individually fused, 3-wire hospital-grade electrical connectors. Each Sterile-Drier individually tested for 1,350-volt dielectric insulation.

Electrical Safety

CSA C/US certified to CSA C22.2 1010-1 and UL 61010A-1.

Department Layout Planning

Your Olympic representative can help you lay out an efficient decontamination area, including Sterile-Driers, and Olympic Pasteuramics.

Warranty

Olympic Sterile-Driers are covered by a one-year warranty on parts.

Ordering Information

OLYMPIC STERILE-DRIERS

Model 44 - With 9 full-size parts drying trays and tube-holder for 35 tubes. 46.5" x 20.5" x 86" (118 x 52 x 218 cm). 120V~, 60 Hz, 1700 watts.

CAT. NO. 54344

Model 43 - With 3 full-size parts drying trays, 6 half-size parts drying trays, and tube-holder for 35 tubes. 26" x 20.5" x 86" (66 x 52 x 218 cm). 120V~, 60 Hz, 900 watts.

CAT. NO. 54343

NOTE: All Driers available with optional rear doors and additional trays and tube holders.

ACCESSORIES

Bag Drier - For use with any Olympic Sterile-Drier manufactured since 1984.

CAT. NO. 54355

*For high-level disinfection,
combine pasteurization
with bio-clean drying*

OLYMPIC PASTEURMATIC™ 3000/3500



Washes & pasteurizes in one completely automatic cycle, with minimal labor & handling.

natus®

Natus Medical Incorporated
1501 Industrial Road • San Carlos, CA 94070 USA
1-800-303-0306 • 1-650-802-0400
www.natus.com

For information or ordering
Call 1-800-303-0306
(toll-free in US/Canada)